


## CHEEK SWAB COLLECTION INSTRUCTIONS

HealthGene Laboratory offers an easy non-invasive way to collect samples for genetic DNA tests.


### Sample Collection Kit

#### Contents

Envelope with 3 brushes

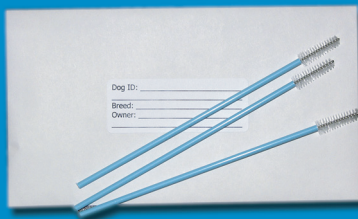

Submission Form

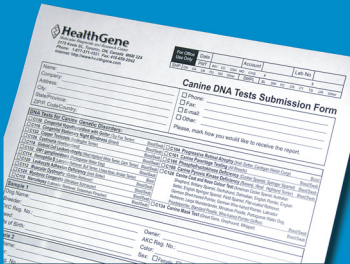

Return Envelope



Information Material



1. Perform the collection on one dog at a time
2. Open the package containing the brushes.
3. Collect genetic material by rubbing a brush 10 times against the inside of one cheek.
4. Repeat step #3 using the same brush on the other cheek.
5. Repeat steps #3 and #4 with the remaining 2 brushes in the package.
6. Enclose the brushes into the paper envelope provided and seal it.
7. Write dog's ID on the label.
8. Fill-in the submission form
9. Place the collected samples, submission form, and payment into the return envelope provided. (\*)
10. Send the package by regular mail. (\*\*)

(\*) - The submission form and the return envelope can be used for multiple sample submission.

(\*\*) - For international shipment, please use any express mail service.

